



how to use **BENDANA**

mod. 001 CONTOURED ABDOMEN AND CHEST



PATENTED




While the animal is still sedated, with a soft meter measure the circumference of the chest




From the taken measurement subtract about 20% and based on the obtained value choose the appropriate BENDANA size




Decide whether to insert two or four legs, then measure the length of the animal up to where the protection is needed (lower abdomen or tail)

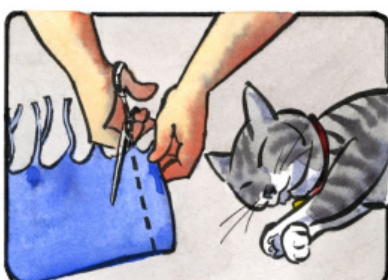


If you want to insert only two legs, shorten the protection by removing one or more fabric fringes

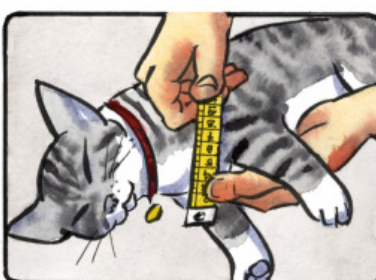

Measure the width of the chest between the two legs

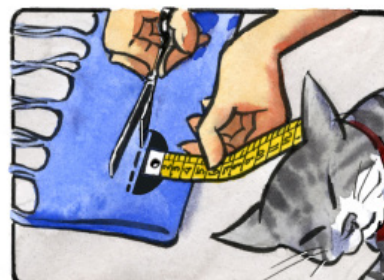

Fold the protection in two and mark 1/2 of the chest measure. If needed, widen the half-moon holes for the legs

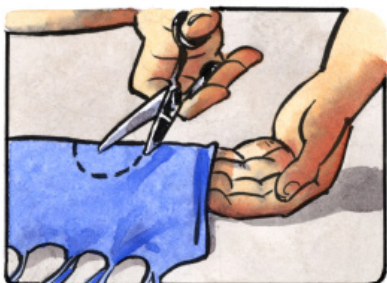

Always with folded protection, if needed, create a hole for male genitals



Insert the legs into the already prepared openings



Pass BENDANA under the animal's side and lace it up on the back



Upon awakening, the animal will be perfectly at ease and protected thanks to BENDANA!

- Removal of cutaneous masses and nodules
- Abdominal surgery
- Thoracic surgery
- Ovarian-hysterectomy-ovariectomy - female sterilization
- Orchiectomy / male sterilization

CONTOURED ABDOMEN AND CHEST, supplied in nine sizes, it is recommended for the SURGERY TYPES mentioned above

- Widespread lesions of various origins
- Licking wounds
- Bite wounds
- Dermatitis (e.g. hot spots, atopy...)

... and for these COMMON PATHOLOGIES



BENDANA

Post-operative protection for veterinary use for pets

T +39 0444 550361 F +39 0444 550268 Arcugnano (Vicenza)- ITALIA

info@bendana.it - www.bendana.it - P. IVA IT03652540240 - n. R.E.A. VI-342637






BENDANA


It is a post-operative protection for veterinary use, designed to completely cover the dressings in a safe and lasting way, leaving the animal absolute freedom of movement and eliminating any annoyance caused by the use of plasters, velcro and so on. It is a very simple to use elastic bandage, adaptable in a few steps to any breed and size, easy to remove even for routine checks during convalescence. It is made of natural jersey fabric ($\geq 90\%$ cotton e $5\div 10\%$ elastane). Produced in various models, in different shapes and sizes depending on the part of the body to be protected, it is supplied in two solutions: as shaped articles ready for use and, for the most experts, as a continuous straight protection, to be cut as required.

PATENTED PRODUCT

HOW TO TAKE THE MEASUREMENTS

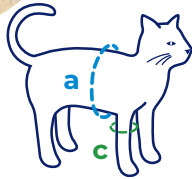


- a chest circumference
-20% approximately= **A**
- c circonferenza omero = **C**

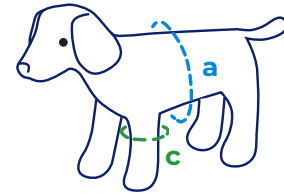

HOW TO CHOOSE THE SIZES (MOD. 001)


CONTOURED ABDOMEN AND CHEST

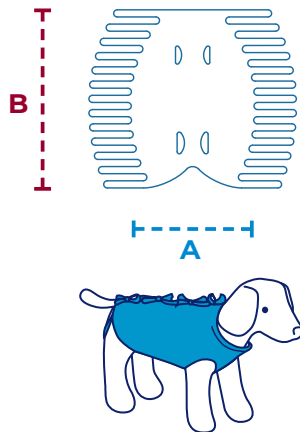

Item	A cm	B cm	size	weight
001.20	20	32	XXXXS	1 - 3
001.25	25	37	XXXS	3 - 5
001.30	30	44	XXS	5 - 7
001.35	35	50	XS	7 - 10
001.40	40	58	S	10 - 15
001.45	45	65	M	15 - 20
001.55	55	80	L	20 - 30
001.65	65	95	XL	30 - 40
001.75	75	110	XXL	40 - 50

All models can be used for dogs and cats equally


12 PRODUCED MODELS

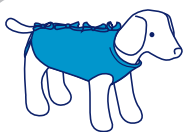

mod. 001
CONTOURED
ABDOMEN AND
CHEST
protection



mod. 101
STRAIGHT
ABDOMEN AND
CHEST
protection

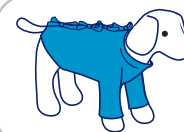


mod. 011
ABDOMEN, CHEST
and FORELEGS
protection



mod. 021
ABDOMEN, CHEST
and HIND LEGS
protection



mod. 031
ABDOMEN, CHEST
AND ALL LEGS
protection

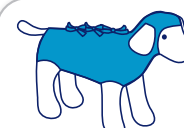

mod. 041
CONTOURED
HEAD, NECK, AND
ABDOMEN
protection




mod. 051
HALTER
protection




mod. 052
ELECTRODES
HOLDING
protection



mod. 061
FORELEGS protection
WITH LACES

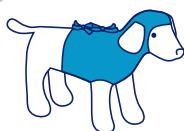

mod. 071
HEAD, NECK,
CHEST
protection

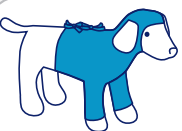

mod. 081
HEAD, NECK, CHEST,
FORELEGS
protection



mod. 091
TOTAL
protection

The complete catalog, price list and order form are available on the website

www.bendana.it