

mod. 011

how to use **BENDANA**


**ABDOMEN, CHEST AND FORELEGS**



PATENTED



While the animal is still sedated, with a soft meter measure the circumference of the chest

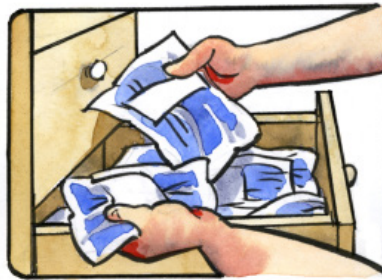


From the taken measurement subtract about 20% and based on the obtained value choose the appropriate BENDANA size



Measure the width of the belly between the two legs



Fold the protection in two and mark 1/2 of the taken measurement; then, if necessary, widen the cuts already pre-arranged

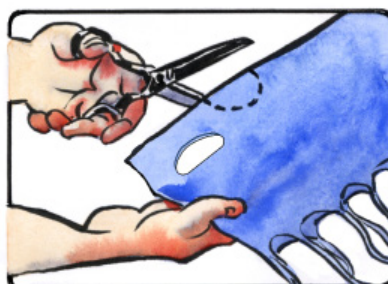

Always with folded protection, if needed, create a hole for male genitals



Insert the front legs into the tubular elements



Insert the hind legs in the holes previously create



Pass BENDANA under the animal's side and lace it up on the back



Upon awakening, the animal will be perfectly at ease and protected thanks to BENDANA!

- Removal of cutaneous masses and nodules
- Abdominal surgery (e.g. enterotomy, splenectomy...)
- Shoulder/elbow surgery (e.g. dysplasias, OCD...)
- Thoracic surgery (e.g. lobectomy, pericardiectomy...)
- Ovarian-hysterectomy - ovariectomy - female sterilization - mastectomy

- Widespread lesions of various origins
- Axillary wounds
- Licking wounds
- Bite wounds
- Dermatitis (e.g. hot spots, atopy...)

ABDOMEN, CHEST AND FORELEGS, supplied in nine sizes, it is recommended for the SURGERY TYPES mentioned above

... and for these COMMON PATHOLOGIES



**BENDANA**

**Post-operative protection for veterinary use for pets**

T +39 0444 550361 F +39 0444 550268 Arcugnano (Vicenza)- ITALIA

info@bendana.it - www.bendana.it - P. IVA IT03652540240 - n. R.E.A. VI-342637






# BENDANA


It is a post-operative protection for veterinary use, designed to completely cover the dressings in a safe and lasting way, leaving the animal absolute freedom of movement and eliminating any annoyance caused by the use of plasters, velcro and so on. It is a very simple to use elastic bandage, adaptable in a few steps to any breed and size, easy to remove even for routine checks during convalescence. It is made of natural jersey fabric ( $\geq 90\%$  cotton e  $5\div 10\%$  elastane). Produced in various models, in different shapes and sizes depending on the part of the body to be protected, it is supplied in two solutions: as shaped articles ready for use and, for the most experts, as a continuous straight protection, to be cut as required.

PATENTED PRODUCT


## HOW TO TAKE THE MEASUREMENTS

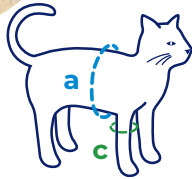

- a** chest circumference  
-20% approximately= **A**
- c** circonferenza omero = **C**

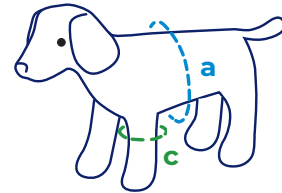

## HOW TO CHOOSE THE SIZES (MOD. 011)

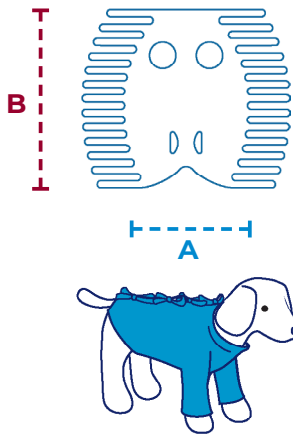


### ABDOMEN, CHEST AND FORELEGS

Item	A cm	B cm	size	weight
001.20	20	32	XXXXS	1 - 3
001.25	25	37	XXXS	3 - 5
001.30	30	44	XXS	5 - 7
001.35	35	50	XS	7 - 10
001.40	40	58	S	10 - 15
001.45	45	65	M	15 - 20
001.55	55	80	L	20 - 30
001.65	65	95	XL	30 - 40
001.75	75	110	XXL	40 - 50

All models can be used for dogs and cats equally


## 12 PRODUCED MODELS

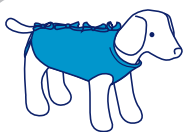

**mod. 001**  
CONTOURED  
ABDOMEN AND  
CHEST  
protection



**mod. 101**  
STRAIGHT  
ABDOMEN AND  
CHEST  
protection

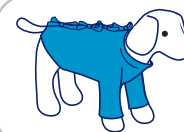


**mod. 011**  
ABDOMEN, CHEST  
and FORELEGS  
protection



**mod. 021**  
ABDOMEN, CHEST  
and HIND LEGS  
protection



**mod. 031**  
ABDOMEN, CHEST  
AND ALL LEGS  
protection

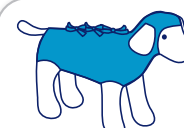

**mod. 041**  
CONTOURED  
HEAD, NECK, AND  
ABDOMEN  
protection




**mod. 051**  
HOLTER  
protection




**mod. 052**  
ELECTRODES  
HOLDING  
protection



**mod. 061**  
FORELEGS protection  
WITH LACES

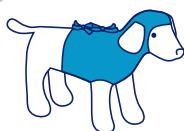

**mod. 071**  
HEAD, NECK,  
CHEST  
protection

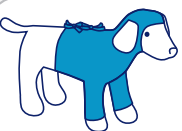

**mod. 081**  
HEAD, NECK, CHEST,  
FORELEGS  
protection



**mod. 091**  
TOTAL  
protection

The complete catalog, price list and order form are available on the website

[www.bendana.it](http://www.bendana.it)