


# how to use **BENDANA**

## mod. 041 CONTOURED HEAD, NECK AND ABDOMEN




PATENTED

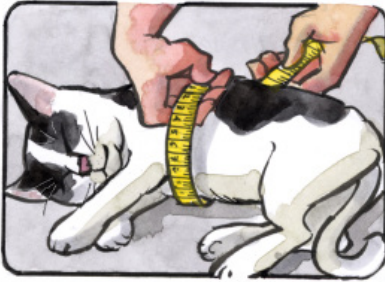


While the animal is still sedated, with a soft meter measure the circumference of the chest




Depending on the measurement (approximately -20%) choose the appropriate BENDANA size

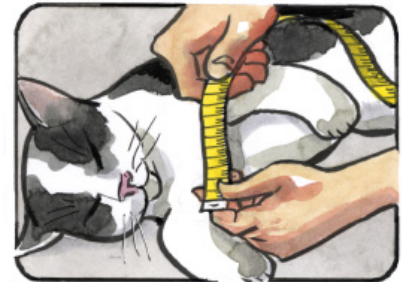


Measure the width of the chest between the two legs

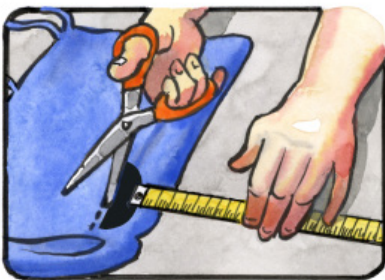


Fold the protection in half and mark 1/2 of the size of the chest; then, if needed, widen the holes for the legs

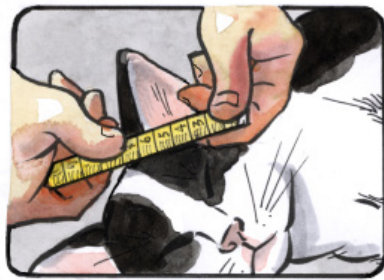


Check the width of the ears and, if necessary, widen the two half-moon holes

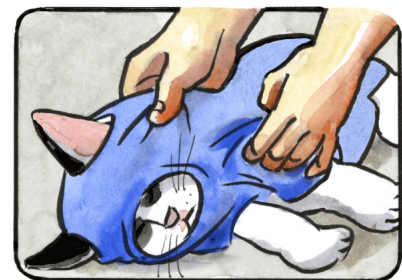


Insert the protection from the head leaving the ears free

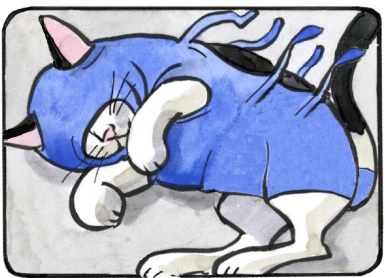


Insert the legs into the holes, widening them if needed

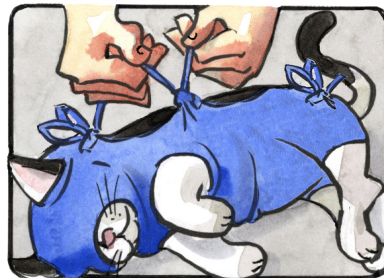

Lace Bendana up on the back

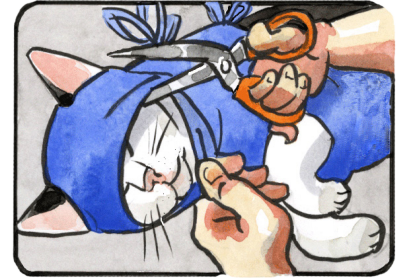

Fold and cut the excess protection, uncovering the snout and freeing the eyes

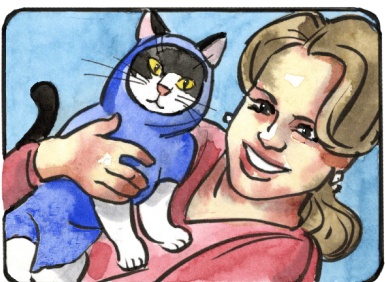

Upon awakening, the animal will be perfectly at ease and protected thanks to BENDANA!

- Abdominal surgery (e.g. enterotomy, splenectomy...)
- Thoracic surgery (e.g. lobectomy, pericardiotomy...)
- Head/neck surgery (e.g. bite wounds, thyroidectomy...)
- Orchiectomy/male sterilization
- Ovarian-hysterectomy - ovariectomy - female sterilization - mastectomy

CONTOURED HEAD, NECK AND ABDOMEN, supplied in nine sizes, it is recommended for the SURGERY TYPES mentioned above

- Widespread lesions of various origins
- Licking wounds
- Bite wounds
- Dermatitis (e.g. hot spots, atopy...)

.... and for these COMMON PATHOLOGIES



**BENDANA**

**Post-operative protection for veterinary use for pets**

T +39 0444 550361 F +39 0444 550268 Arcugnano (Vicenza)- ITALIA

info@bendana.it - www.bendana.it - P. IVA IT03652540240 - n. R.E.A. VI-342637






# BENDANA

It is a post-operative protection for veterinary use, designed to completely cover the dressings in a safe and lasting way, leaving the animal absolute freedom of movement and eliminating any annoyance caused by the use of plasters, velcro and so on. It is a very simple to use elastic bandage, adaptable in a few steps to any breed and size, easy to remove even for routine checks during convalescence. It is made of natural jersey fabric ( $\geq 90\%$  cotton e  $5\div 10\%$  elastane). Produced in various models, in different shapes and sizes depending on the part of the body to be protected, it is supplied in two solutions: as shaped articles ready for use and, for the most experts, as a continuous straight protection, to be cut as required.


PATENTED PRODUCT




## HOW TO TAKE THE MEASUREMENTS

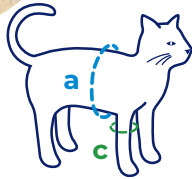

- a** chest circumference  
-20% approximately= **A**
- c** circonferenza omero = **C**

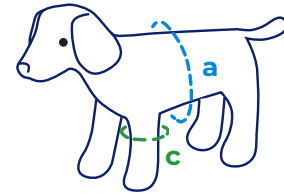

## HOW TO CHOOSE THE SIZES (MOD. 041)

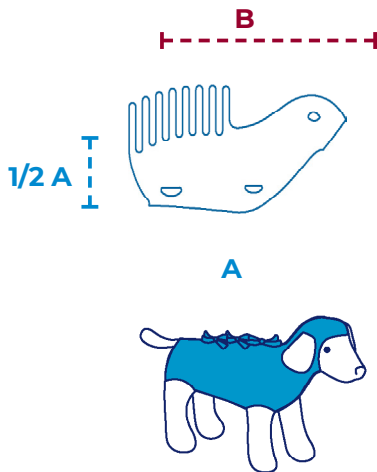

### CONTOURED HEAD, NECK AND ABDOMEN

Item	A cm	B cm	size	weight
001.20	20	32	XXXXS	1 - 3
001.25	25	37	XXXS	3 - 5
001.30	30	44	XXS	5 - 7
001.35	35	50	XS	7 - 10
001.40	40	58	S	10 - 15
001.45	45	65	M	15 - 20
001.55	55	80	L	20 - 30
001.65	65	95	XL	30 - 40
001.75	75	110	XXL	40 - 50

All models can be used for dogs and cats equally

## 12 PRODUCED MODELS

	<b>mod. 001</b> CONTOURED ABDOMEN AND CHEST protection		<b>mod. 101</b> STRAIGHT ABDOMEN AND CHEST protection		<b>mod. 011</b> ABDOMEN, CHEST and FORELEGS protection
	<b>mod. 021</b> ABDOMEN, CHEST and HIND LEGS protection		<b>mod. 031</b> ABDOMEN, CHEST AND ALL LEGS protection		<b>mod. 041</b> CONTOURED HEAD, NECK, AND ABDOMEN protection
	<b>mod. 051</b> HOLTER protection		<b>mod. 052</b> ELECTRODES HOLDING protection		<b>mod. 061</b> FORELEGS protection WITH LACES
	<b>mod. 071</b> HEAD, NECK, CHEST protection		<b>mod. 081</b> HEAD, NECK, CHEST, FORELEGS protection		<b>mod. 091</b> TOTAL protection

The complete catalog, price list and order form are available on the website

[www.bendana.it](http://www.bendana.it)