



how to use **BENDANA**


mod. 052 ELECTRODES HOLDING PROTECTION



PATENTED




Spread material: 1. electro-medical devices, 2. electrode cover, 3. Holter holding protection

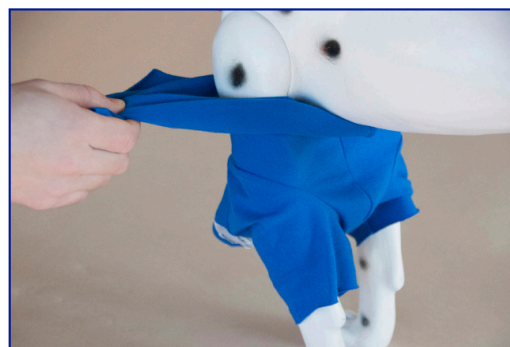

Apply the cover: inserting the legs first and then the head



Electrode fixing phase

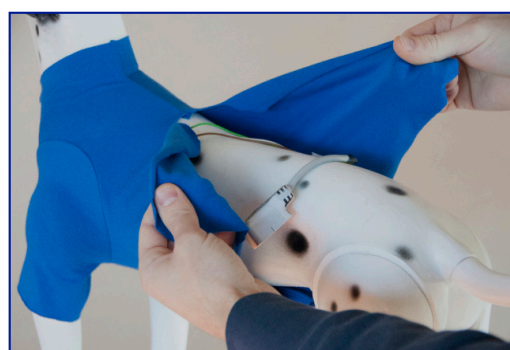

Electrode fixing phase



Pass the connector within the hole of the cover and let the two flaps overlap



After having well overlapped the two flaps, fix them with at least two plaster strips



Insert the Holter holding protection: first the front legs and then the head



Place the protection on the back of the animal centering it inasmuch as possible



BENDANA

Post-operative protection for veterinary use for pets

T +39 0444 550361 F +39 0444 550268 Arcugnano (Vicenza)- ITALIA

info@bendana.it - www.bendana.it - P. IVA IT03652540240 - n. R.E.A. VI-342637






BENDANA


It is a post-operative protection for veterinary use, designed to completely cover the dressings in a safe and lasting way, leaving the animal absolute freedom of movement and eliminating any annoyance caused by the use of plasters, velcro and so on. It is a very simple to use elastic bandage, adaptable in a few steps to any breed and size, easy to remove even for routine checks during convalescence. It is made of natural jersey fabric ($\geq 90\%$ cotton e $5\div 10\%$ elastane). Produced in various models, in different shapes and sizes depending on the part of the body to be protected, it is supplied in two solutions: as shaped articles ready for use and, for the most experts, as a continuous straight protection, to be cut as required.

PATENTED PRODUCT

HOW TO TAKE THE MEASUREMENTS


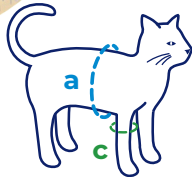


- a** chest circumference
-20% approximately= **A**
- c** circonferenza omero = **C**

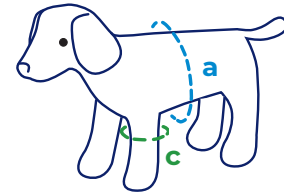

HOW TO CHOOSE THE SIZES (MOD. 052)

ELECTRODES HOLDING

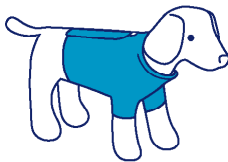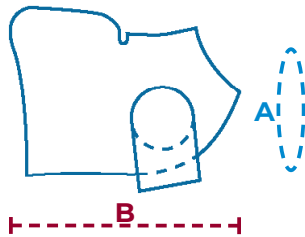

Item	A cm	B cm	size	weight
001.30	30	44	XXS	5 - 7
001.35	35	50	XS	7 - 10
001.40	40	58	S	10 - 15
001.45	45	65	M	15 - 20
001.55	55	80	L	20 - 30
001.65	65	95	XL	30 - 40
001.75	75	110	XXL	40 - 50

All models can be used for dogs and cats equally


12 PRODUCED MODELS



mod. 001
CONTOURED
ABDOMEN AND
CHEST
protection



mod. 101
STRAIGHT
ABDOMEN AND
CHEST
protection

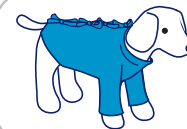


mod. 011
ABDOMEN, CHEST
and FORELEGS
protection



mod. 021
ABDOMEN, CHEST
and HIND LEGS
protection



mod. 031
ABDOMEN, CHEST
AND ALL LEGS
protection

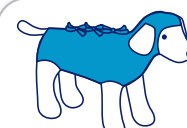

mod. 041
CONTOURED
HEAD, NECK, AND
ABDOMEN
protection




mod. 051
HOLTER
protection




mod. 052
ELECTRODES
HOLDING
protection



mod. 061
FORELEGS protection
WITH LACES

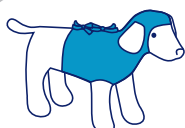

mod. 071
HEAD, NECK,
CHEST
protection

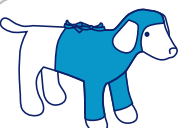

mod. 081
HEAD, NECK, CHEST,
FORELEGS
protection



mod. 091
TOTAL
protection

The complete catalog, price list and order form are available on the website

www.bendana.it