

## how to use BENDANA 💌 FORELEGS WITH LACES





While the animal is still sedated, with a soft meter measure the circumference of the humerus



Depending on the measurement, choose the appropriate BENDANA size



Fold BENDANA by half, mark 1/2 of the detected measurement and shorten the protection if it is too long



Upon awakening, the animal will be perfectly at ease and protected thanks to BENDANA!



Insert the legs

- Removal of cutaneous masses and nodules
- Shoulder/elbow surgery (e.g. dysplasias, OCD...)
- Thoracic surgery (e.g. lobectomy, pericardiotomy...)

FORELEGS WITH LACES, supplied in nine sizes, it is recommended for the SURGERY TYPES mentioned above



Measure the length of the legs added to the width of the animal's chest (passing under the chest)



Pass BENDANA under the animal's side and lace it up on the back

| • Widespread lesions of various ori- |       |     |        |
|--------------------------------------|-------|-----|--------|
| gins                                 |       |     |        |
| <ul> <li>Axillary wounds</li> </ul>  |       |     |        |
| <ul> <li>Licking wounds</li> </ul>   |       |     |        |
| • Bite wounds                        |       |     |        |
| • Dermatitis                         | (e.g. | hot | spots, |
| atopy)                               |       |     | •      |
|                                      |       |     |        |
| and for these COMMON                 |       |     |        |

PATHOLOGIES




Post-operative protection for veterinary use for pets T +39 0444 550361 F +39 0444 550268 Arcugnano (Vicenza)- ITALIA BENDANA info@bendana.it - www.bendana.it - P. IVA IT03652540240 - n. R.E.A. VI-342637






It is a post-operative protection for veterinary use, designed to completely cover the dressings in a safe and lasting way, leaving the animal absolute freedom of movement and eliminating any annoyance caused by the use of plasters, velcro and so on. It is a very simple to use elastic bandage, adaptable in a few steps to any breed and size, easy to remove even for routine checks during convalescence. It is made of natural jersey fabric ( $\geq$  90% cotton e 5÷10% elastane). Produced in various models, in different shapes and sizes depending on the part of the body to be protected, it is supplied in two solutions: as shaped articles ready for use and, for the most experts, as a continuous straight protection, to be cut as required.

PATENTED PRODUCT

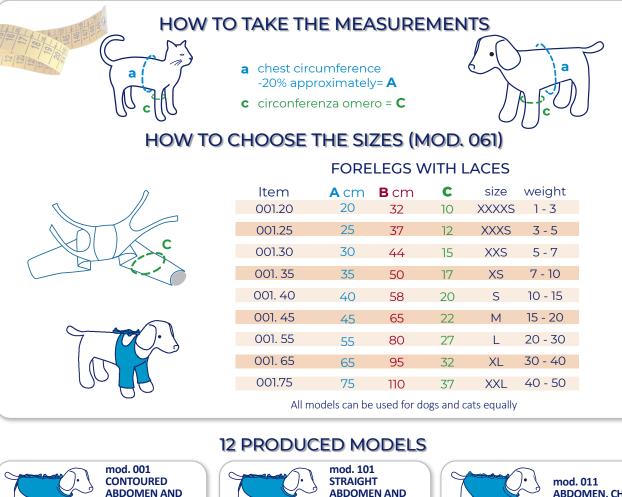

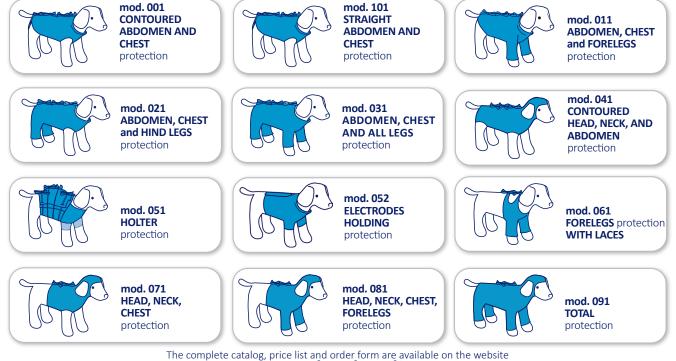

www.bendana.it