

mod. 071

how to use **BENDANA** HEAD, NECK AND CHEST



PATENTED




While the animal is still sedated, with a soft meter measure the circumference of the chest



Depending on the measurement (approximately -20%) choose the appropriate BENDANA size



Measure the width of the chest between the two legs

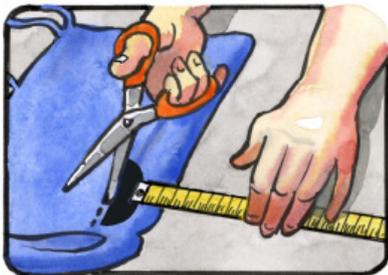

Fold the protection by half and mark 1/2 of the size of the chest; then, if needed, widen the holes for the legs



Check the width of the ears and, if necessary, widen the two half-moon holes



Insert the protection from the head leaving the ears free



Insert the legs into the holes, widening them if needed



Lace Bendana up on the back



Fold and cut the excess protection, uncovering the snout and freeing the eyes



Upon awakening, the animal will be perfectly at ease and protected thanks to BENDANA!

- Removal of cutaneous masses and nodules
- Thoracic surgery (e.g. lobectomy, pericardiectomy...)
- Head/neck surgery (e.g. bite wounds, thyroidectomy...)

HEAD, NECK and CHEST, supplied in nine sizes, it is recommended for the SURGERY TYPES mentioned above

- Widespread lesions of various origins
- Licking wounds
- Bite wounds
- Dermatitis (e.g. hot spots, atopy...)

.... and for these COMMON PATHOLOGIES



BENDANA

Post-operative protection for veterinary use for pets

T +39 0444 550361 F +39 0444 550268 Arcugnano (Vicenza)- ITALIA

info@bendana.it - www.bendana.it - P. IVA IT03652540240 - n. R.E.A. VI-342637



MADE IN ITALY




BENDANA


It is a post-operative protection for veterinary use, designed to completely cover the dressings in a safe and lasting way, leaving the animal absolute freedom of movement and eliminating any annoyance caused by the use of plasters, velcro and so on. It is a very simple to use elastic bandage, adaptable in a few steps to any breed and size, easy to remove even for routine checks during convalescence. It is made of natural jersey fabric ($\geq 90\%$ cotton e $5\div 10\%$ elastane). Produced in various models, in different shapes and sizes depending on the part of the body to be protected, it is supplied in two solutions: as shaped articles ready for use and, for the most experts, as a continuous straight protection, to be cut as required.

PATENTED PRODUCT


HOW TO TAKE THE MEASUREMENTS

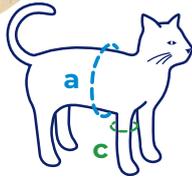

- a** chest circumference
-20% approximately= **A**
- c** circonferenza omero = **C**

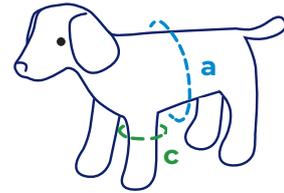


HOW TO CHOOSE THE SIZES (MOD. 071)

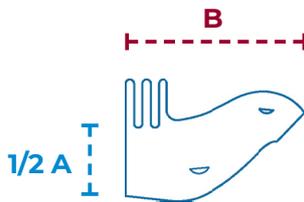


HEAD, NECK AND CHEST

Item	A cm	B cm	size	weight
001.20	20	32	XXXXS	1 - 3
001.25	25	37	XXXS	3 - 5
001.30	30	44	XXS	5 - 7
001.35	35	50	XS	7 - 10
001.40	40	58	S	10 - 15
001.45	45	65	M	15 - 20
001.55	55	80	L	20 - 30
001.65	65	95	XL	30 - 40
001.75	75	110	XXL	40 - 50

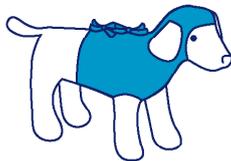

All models can be used for dogs and cats equally

12 PRODUCED MODELS

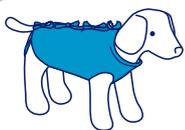

mod. 001
CONTOURED
ABDOMEN AND
CHEST
protection



mod. 101
STRAIGHT
ABDOMEN AND
CHEST
protection




mod. 011
ABDOMEN, CHEST
and FORELEGS
protection



mod. 021
ABDOMEN, CHEST
and HIND LEGS
protection



mod. 031
ABDOMEN, CHEST
AND ALL LEGS
protection

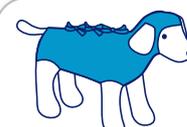

mod. 041
CONTOURED
HEAD, NECK, AND
ABDOMEN
protection




mod. 051
HOLTER
protection



mod. 052
ELECTRODES
HOLDING
protection



mod. 061
FORELEGS protection
WITH LACES

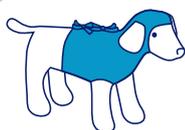

mod. 071
HEAD, NECK,
CHEST
protection



mod. 081
HEAD, NECK, CHEST,
FORELEGS
protection



mod. 091
TOTAL
protection

The complete catalog, price list and order form are available on the website

www.bendana.it