

While the animal is still sedated, with a soft meter measure the circumference of the chest



Depending on the measurement (approximately -20%) choose the appropriate BENDANA size



Measure the length of the legs added to the width of the animal's chest (passing under the chest)




Mark half of the just measured length and shorten the tubular elements accordingly




Check the width of the ears and, if necessary, widen the two half-moon holes



Insert the protection from the head leaving the ears free

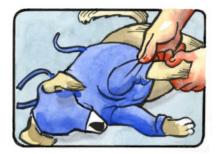

Insert the front legs into the tubular

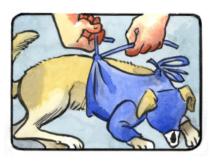

Lace Bendana up on the back



Cut the excess protection and discover the whole snout, or create two holes for the eyes



Upon awakening, the animal will be perfectly at ease and protected thanks to BENDANA!

- Removal of cutaneous masses and nodules
- Shoulder/elbow surgery (e.g. dysplasias, OCD...)
- Thoracic surgery (e.g. lobectomy, pericardiotomy...)
- Head/neck surgery (e.g. bite wounds, thyroidectomy...)

HEAD, NECK, CHEST AND FORELEGS, supplied in nine sizes, it is recommended for the SURGERY

- Widespread lesions of various origins
- Axillary wounds
- Licking wounds
- Bite wounds
- Dermatitis (e.g. hot spots, atopy...)
- Widespread burns

these COMMON and for PATHOLOGIES:



TYPES mentioned above




It is a post-operative protection for veterinary use, designed to completely cover the dressings in a safe and lasting way, leaving the animal absolute freedom of movement and eliminating any annoyance caused by the use of plasters, velcro and so on. It is a very simple to use elastic bandage, adaptable in a few steps to any breed and size, easy to remove even for routine checks during convalescence. It is made of natural jersey fabric (\geq 90% cotton e 5÷10% elastane). Produced in various models, in different shapes and sizes depending on the part of the body to be protected, it is supplied in two solutions: as shaped articles ready for use and, for the most experts, as a continuous straight protection, to be cut as required.

PATENTED PRODUCT




HOW TO TAKE THE MEASUREMENTS

- a chest circumference-20% approximately= A
- c circonferenza omero = C

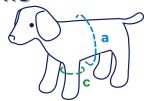

HOW TO CHOOSE THE SIZES (MOD. 081)

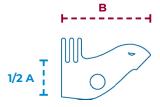



HEAD, NECK, CHEST, FORELEGS

ltem	A cm	B cm	size	weight
001.20	20	32	XXXXS	1 - 3
001.25	25	37	XXXS	3 - 5
001.30	30	44	XXS	5 - 7
001. 35	35	50	XS	7 - 10
001. 40	40	58	S	10 - 15
001. 45	45	65	М	15 - 20
001. 55	55	80	L	20 - 30
001. 65	65	95	XL	30 - 40
001.75	75	110	XXL	40 - 50

All models can be used for dogs and cats equally

12 PRODUCED MODELS



mod. 001 CONTOURED ABDOMEN AND CHEST protection



mod. 101 STRAIGHT ABDOMEN AND CHEST protection



mod. 011
ABDOMEN, CHEST
and FORELEGS
protection



mod. 021 ABDOMEN, CHEST and HIND LEGS protection



mod. 031
ABDOMEN, CHEST
AND ALL LEGS
protection



mod. 041 CONTOURED HEAD, NECK, AND ABDOMEN protection



mod. 051 HOLTER protection



mod. 052 ELECTRODES HOLDING protection



mod. 061
FORELEGS protection
WITH LACES



mod. 071 HEAD, NECK, CHEST protection



mod. 081 HEAD, NECK, CHEST, FORELEGS protection



mod. 091 TOTAL protection