

how to use BENDANA 💌 TOTAL PROTECTION





While the animal is still sedated, with a soft meter measure the circumference of the chest



Mark half of the just measured length and shorten the tubular elements accordingly



Insert the front and rear legs into the tubular elements



Upon awakening, the animal will be perfectly at ease and protected thanks to BENDANA!



Depending on the measurement (approximately -20%) choose the appropriate BENDANA size



Check the width of the ears and, if necessary, widen the two half-moon holes




Lace Bendana up on the back

- Removal of cutaneous masses and nodules
- Abdominal surgery (e.g. enterotomy, splenectomy...)
- Shoulder/elbow surgery (e.g. dysplasias, OCD...)
- Thoracic surgery (e.g. lobectomy, pericardiotomy...)
- Hip/knee/pelvis surgery (e.g. dysplasias, dislocations, traumas...)
- Head/neck surgery (e.g. bite wounds, thyroidectomy...)
- Orchiectomy/male sterilization
- Ovarian-hysterectomy ovariectomy -
- female sterilization mastectomy

TOTAL PROTECTION, supplied in nine sizes, it is recommended for the SURGERY TYPES mentioned above



Measure the length of the legs added to the width of the animal's chest (passing under the chest)

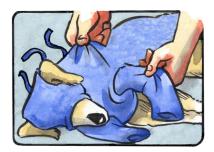

Insert the protection from the head leaving the ears free



Cut the excess protection and discover the whole snout, or create two holes for the eyes

- Widespread lesions of various origins
- Axillary wounds
- Licking wounds
- Bite wounds
- Dermatitis (e.g. hot spots, atopy...)
- Widespread burns

... and for these COMMON PATHOLOGIES




Post-operative protection for veterinary use for pets T +39 0444 550361 F +39 0444 550268 Arcugnano (Vicenza)- ITALIA BENDANA info@bendana.it - www.bendana.it - P. IVA IT03652540240 - n. R.E.A. VI-342637





It is a post-operative protection for veterinary use, designed to completely cover the dressings in a safe and lasting way, leaving the animal absolute freedom of movement and eliminating any annoyance caused by the use of plasters, velcro and so on. It is a very simple to use elastic bandage, adaptable in a few steps to any breed and size, easy to remove even for routine checks during convalescence. It is made of natural jersey fabric (\geq 90% cotton e 5÷10% elastane). Produced in various models, in different shapes and sizes depending on the part of the body to be protected, it is supplied in two solutions: as shaped articles ready for use and, for the most experts, as a continuous straight protection, to be cut as required.

PATENTED PRODUCT

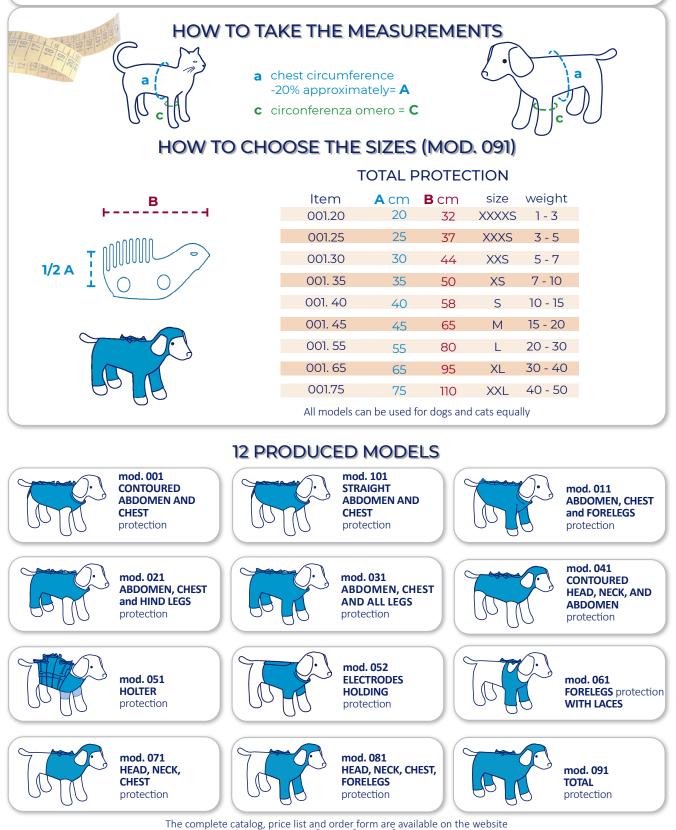

www.bendana.it